KULLANIM KILAVUZU USER GUIDE

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.....

Ürün İlk Kul. Tar.
Date of first use :

Kullanıcı Ad/Soyad
Name&Surname of user:

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(EN) Thank you for purchasing Kaya Safety Conic Stretcher. You will find it a valuable tool that will enable rescues to be performed safer, faster, and easier. Our Conic Stretcher is an entirely different concept in stretcher design. Therefore, potential rescuers should practice and familiarize them selves prior to using it on an actual rescue. This stretcher provides excellent patient support and protection however if a spinal injury is suspected, secure the patient to an approved spinal immobilization device prior to placing on the stretcher. The stretcher must be used with its original accessories provided in the package.

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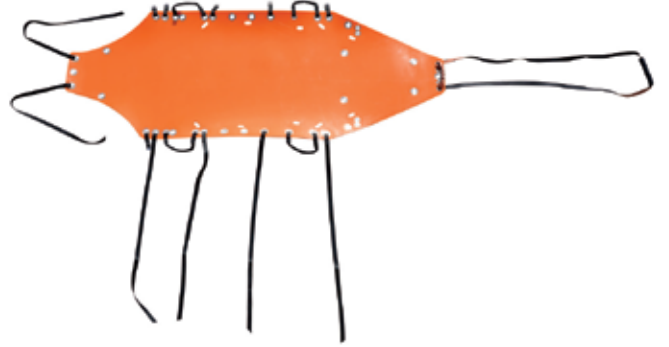
(EN) To Place A Patient On The Stretcher Pull Method (Feet First)

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(EN) Continue dragging until his upper chest is even with the upper cross strap.

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(EN) Secure all buckles and adjust all straps in preparation for transport.

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(EN) Curl foot end of the Stretcher downward to form a ramp and pull foot end straps to the side.

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(EN) Using your forearms to prevent lateral movement of patient's head, grasp the underarms and drag inline with the spine onto the stretcher.

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(EN) Prior to loading the stretcher release all buckles and pull straps to one side.

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(EN) Lift sides of stretcher and fasten the four cross straps to buckles directly opposite to the straps and pull them tight.

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(EN) Fasten foot end buckles and adjust straps.

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(EN) Do not criss-cross straps. It will cause the stretcher to bend in the middle.

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(EN) Insert one end of head strap through angled slot at head end of stretcher.

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(EN) Bring strap under stretcher and through angled slot on opposite side.

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(EN) Equalize straps. Repeat procedure with other strap at foot end of the stretcher.

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(EN) Equalize all four straps and secure ends with K-14/3A large steel locking carabiner. Your stretcher is now ready for horizontal lifting.

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(EN) Prior to hoisting. The head end must be curled over the patient's head and the drag handle webbing tied to the second cross strap to protect patient's head from falling objects.

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(EN) Insert vertical lift sling rope through the grommets above patient's head.

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(EN) For extremely small openings, leave the head end extended. Start threading the rope through the grommets by the patient's shoulders.

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(EN) Continue threading the rope on each side of the stretcher as shown in picture above.

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(EN) Thread the rope through the bottom grommets from the insideout as shown above.

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(EN) Tie the rope in a square knot at the lower end of the stretcher.

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(EN) Pass the ends of the rope through the lower carry handles from the outside-in and tie another square knot.

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(EN) Tie an overhand knot on eachside of the square knot for safety.

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(EN) Hold the pre-tied figure 8 knot in the center of the rope and pass the ends of the rope through the angled lift sling slots from the inside-out.

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(EN) Pass the ends of the rope under the stretcher and through the angled lift sling slots on the opposite side from the outsidein.

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(EN) Tie figure 8 knots in both ends of the rope leaving about 2.5 feet extending up from the foot end slots and shorter at the head end.

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(EN) The rope at the head end must be a bit shorter than the foot end to provide a slightly "head up" attitude for patient's safety and comfort.

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(EN) Lay stretcher out flat with all cross straps connected. Place re - tainer strap with inside of buckle facing up under foot end of stretcher.

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(EN) Starting at the head end, roll the stretcher up as tight as possible. Continue to roll the stretcher using knee to keep the rolled stretcher small.

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(EN) Fasten the preplaced retainer strap to buckle and place stretcher in backpack.

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EN

Guarantee

This product has 10 year guarantee against all material manufacturing defects under proper usage and storage conditions. Guarantee is no more valid if; product is used wrong, tears, cut, corrosion, unauthorised repair of changes on the product, damages caused by accidents.

Certification

* KAYA SAFETY has CE0123 notified body number 89/686/EEC 11 B Quality System certificate and commit to produce each product same as tested sample.

Warning: Read user guide carefully before usage.

Warning

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Adress: GOSB. 1000 Sok. No:1015 Çayırova 41480 KOCAELİ / TÜRKİYE

T: + 90 262 677 19 00

F: + 90 262 677 19 01

e-mail: satis@kayasafety.com

web: www.kayasafety.com

Gebze OSB 1000 Sk. No: 1015 41480

Kocaeli, Turkey

T: + 90 262 677 19 00 F: + 90 262 677 19 01

E: SATIS@KAYASAFETY.COM

KAYASAFETY.COM